



**1** Prior to venipuncture disinfect the puncture site according to protocol and inspect the catheter. Start insertion at an angle you are accustomed to.




**2** Insert catheter until flashback is observed in the chamber. After flashback, advance the entire catheter 3-5 mm. Withdraw steel needle by 1 cm, keeping the catheter in position. Observe flashback in the capillary and thread catheter into the vein.

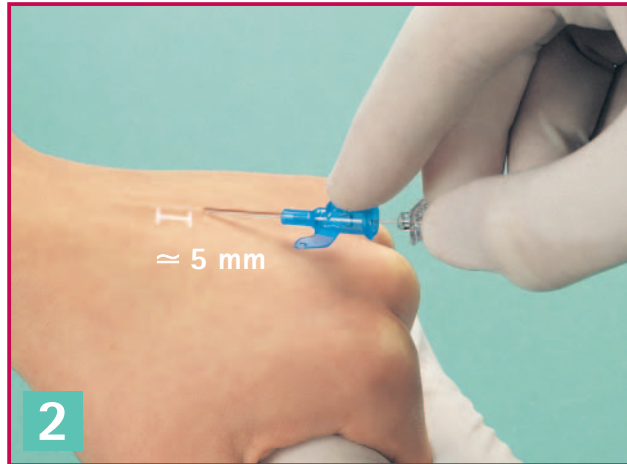

**3** Fix catheter to the skin with the needle still in place. This will minimize spillage of blood.

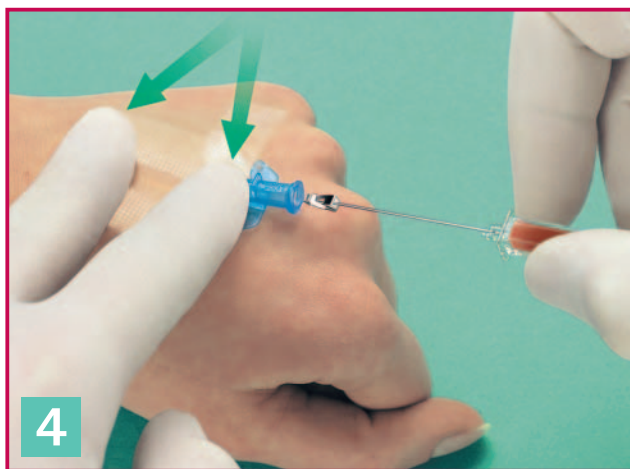

**4** Support the hub by using the V-technique and remove the needle.

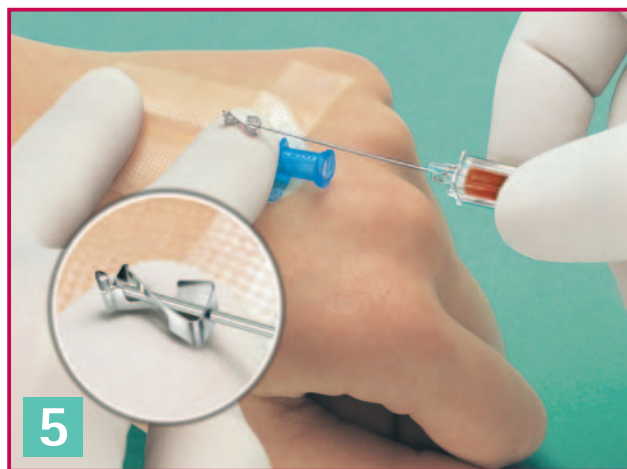


**5** The passive design activates with the safety clip attaching to the exiting needle bevel automatically.



**6** Connect infusion line to catheter hub and cover puncture site with sterile dressing.

#### User benefits

- Easy puncture at a wide range of angles
- Minimum effort of catheter insertion
- Self-activating clip technology – covers needle tip automatically after use

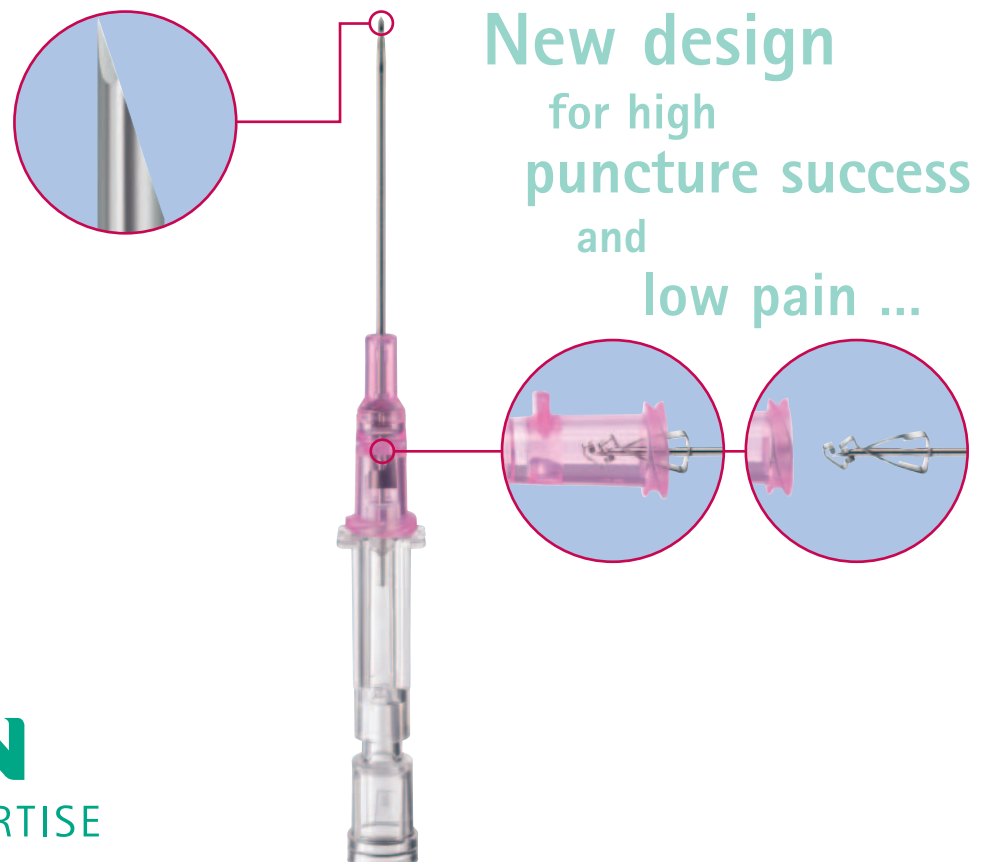

**B | BRAUN**  
SHARING EXPERTISE