

Perifix® Catheter Fixation

To ensure patient safety and comfort



The easy way of fixating a Regional Anesthesia catheter

Perifix® Catheter Fixation

Easy and safe handling




1

Remove paper from the **fixation ring**.




2

Appropriately curl the Regional Anesthesia catheter around the puncture site.

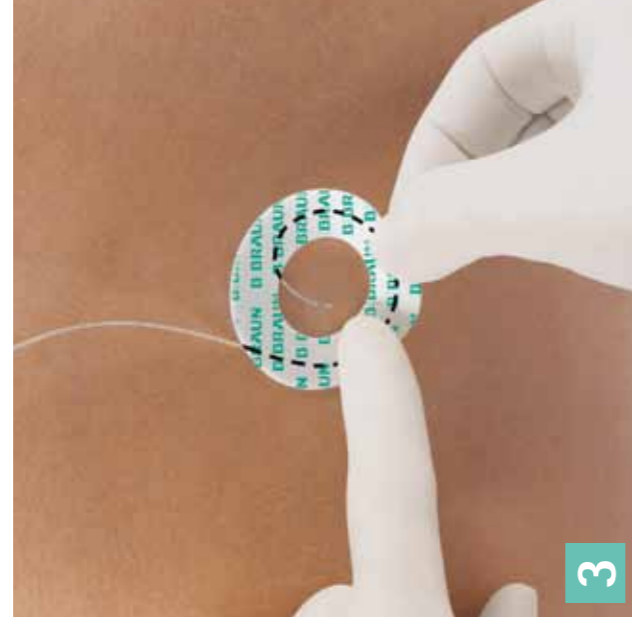

3

Apply the **fixation ring** over the catheter, the catheter is fixed.

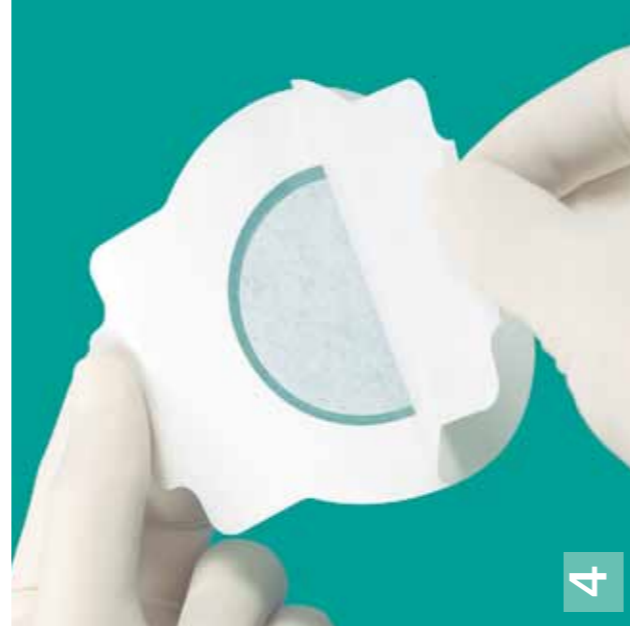


4

Remove paper from **cover foil**.




5

Apply the **cover foil** over the fixation ring.

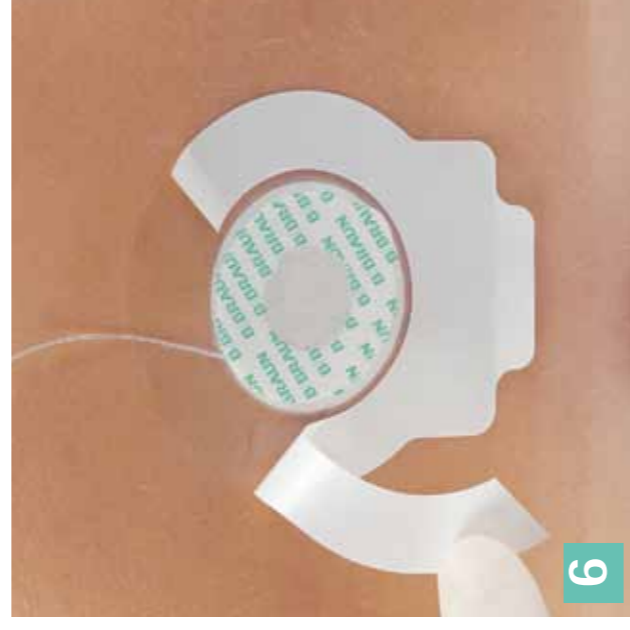

6

Peel off the top paper of the **cover foil**.

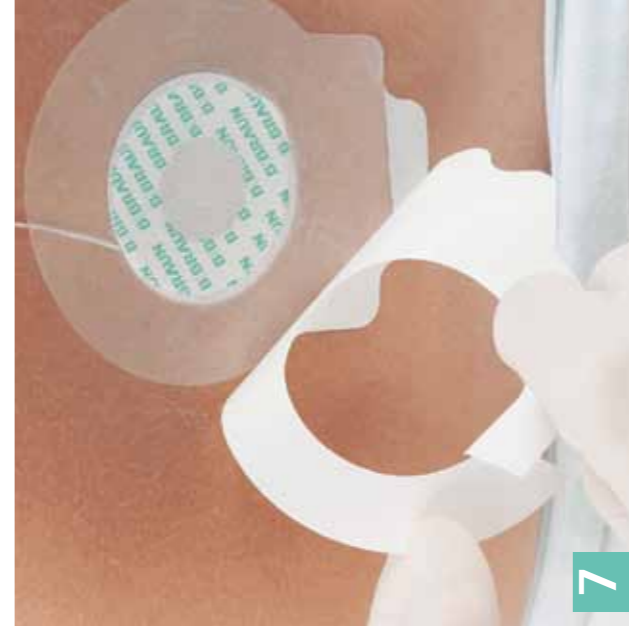

7

Remove the top paper of the **cover foil** completely.



8

Puncture site is protected.

Features and user benefits

- **Fixation ring:**
 - offers a secure and easy fixation to prevent catheter migration
- **Cover foil with non-adhesive center:**
 - provides comfortable protection of puncture site
 - can easily be exchanged without risk of catheter displacement

B | BRAUN
SHARING EXPERTISE

Perifix® Catheter Fixation

The Perifix® Catheter Fixation combines easy handling with enhanced catheter safety.


Secondary catheter migration can be easily avoided. The examination of the puncture site is possible without the risk of re-positioning the catheter during the indwelling time.

All you need for an easy and safe fixation – Perifix® Catheter Fixation.



Easy examination of the puncture site due to the features of the cover foil

Flat design ensures high patient comfort

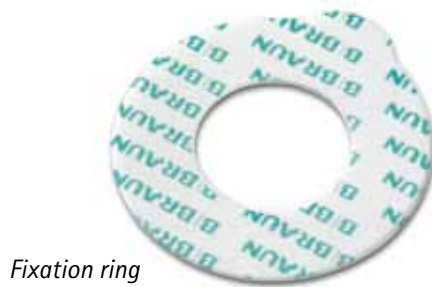

Fixation ring



Cover foil with non-adhesive center

Features and user benefits

- **Fixation ring:** offers a secure and easy fixation to prevent catheter migration
- **Cover foil with non-adhesive center:**
 - provides comfortable protection of puncture site
 - can easily be exchanged without risk of catheter displacement

Description	Code No. (REF)	Sales unit/Pcs.
Perifix® Catheter Fixation	4511200	25
Perifix® Catheter Fixation Cover	4511201	25