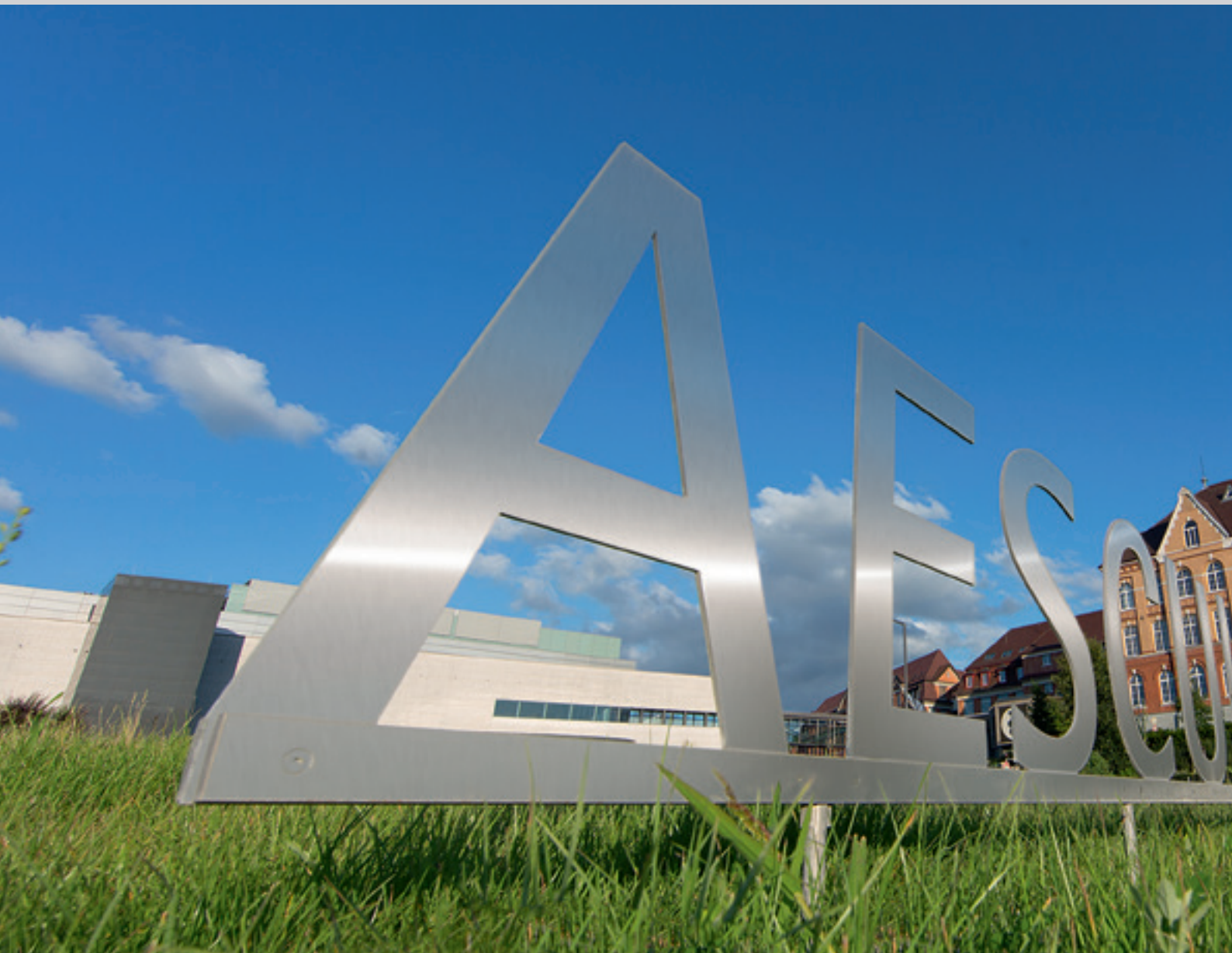
IMPLANTOLOGY

# AESCULAP® DENTAL

COMPETENCE IN DENTISTRY

# AESCULAP® DENTAL IMPLANTOLOGY

COMPETENCE IN DENTISTRY

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with deepening precision and using less and less invasive techniques.

Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

With its new and innovative products, Aesculap® is playing a decisive part in this development. Aesculap's products are developed strictly by the close contact to our customers. This applies as much to new and innovative areas such as microsurgical

instruments for implantology, periodontics and endodontics, as to standard dental instruments in general. Throughout the world, Aesculap® products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available, optimized hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. Aesculap's range of services can help to achieve such improvements.

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The following service components enable us to meet the ever increasing demands of the future:

- our many years of experience and competence in the field of planning and advice,

- the world renowned excellent quality of our products,
- our high level of product availability,
- individual and practice-oriented service concepts to maintain the value of your dental instruments,
- continuous new opportunities for ongoing and further training in the form of the courses offered by the Aesculap® Academy.

With ERGOPLANT, the ergonomically designed instrument for dental implan-

tology, Aesculap® created a real enhancement: ERGOPLANT is complete and unitary productline, which covers all indications, is available to the implantology focused dentist. Where ever necessary, the working tips had been improved for each indication. Due to the ergonomic handle design the user is supported by easiness and grip. Finally the surgeon can concentrate on the patient respectively the surgery. The instrument handling is done intuitively.

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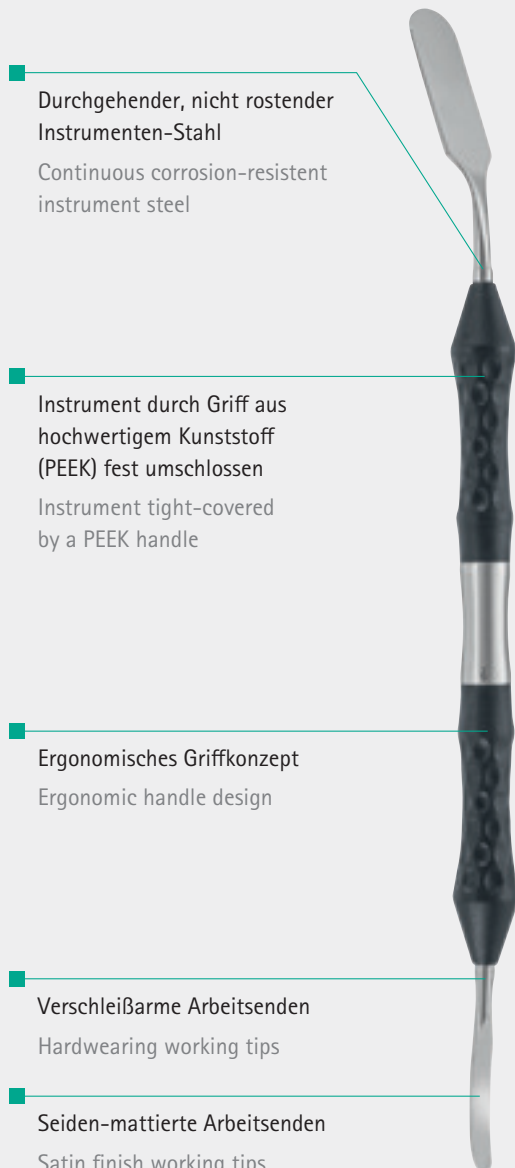
Continuous corrosion-resistant instrument steel

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Instrument tight-covered by a PEEK handle

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In close collaboration with Dr. Dr. Andres Stricker, Freiburg University, an instrument range especially designed for dental implantology has been developed for ergonomics concerning the working tips and handle for any specific indication. ERGOPLANT thus enables working efficiently with minor physical strain under practice conditions. All ERGOPLANT instruments are suitable for steam sterilization 134°C (Autoclav).

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Titanium-microblade holder with round handle, light. Easily rotated and easy to handle for precision incisions.

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Sales unit: **PAK** = Package of 1 piece

Mikrochirurgische Klinge, stainless steel, steril verpackt

Microsurgical blade, stainless steel, sterile packed

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Scalpel blade holder with round handle

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Scalpel blades, vacuum heat treated, sterile packed

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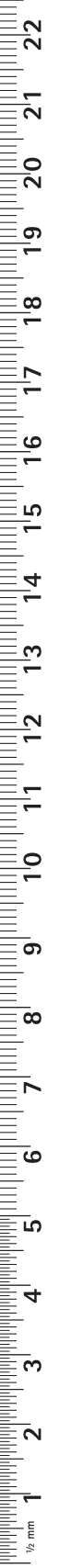
Atraumatic dissecting of the flap by means of the papilla elevator without any damage to the papillae.

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Rounded plate-shaped working tips allow more movement, and significantly increase the radius of mobilisation in frontal and lateral sub-periosteal preparation.

The sharpened edge of the instrument tips also allows sulcular as well as tunnel technique preparation (as with the envelope technique).

The design and angle of the working tips significantly facilitates preparation in convex areas.

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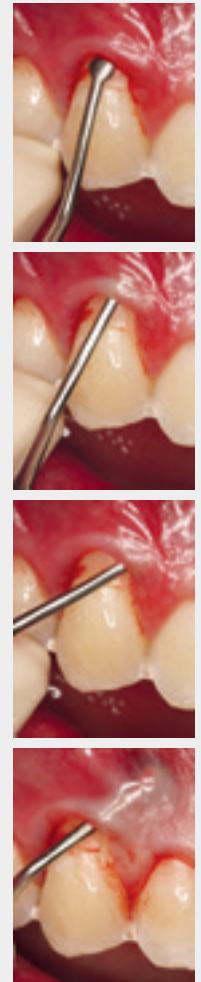
The straight tunnelling instrument is mainly used in the upper jaw, in the frontal and lateral region. This allows large areas of the gingiva to be undermined using both tunnel and sulcular techniques.

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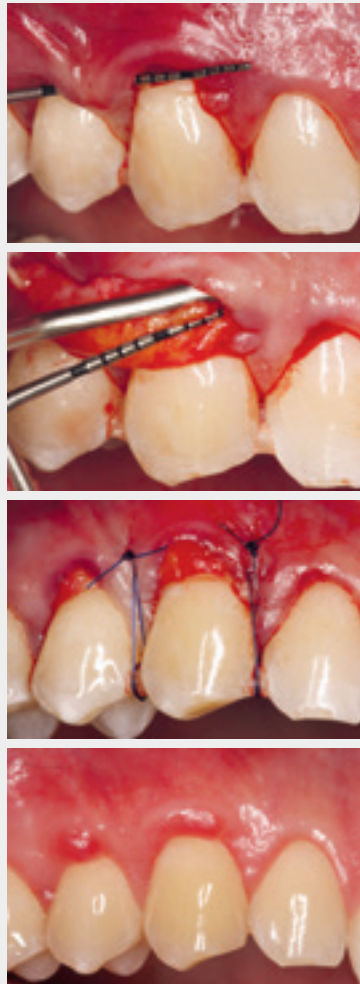
The angled tunnelling instrument is primarily used for preparing lateral, buccal regions. It can also be used distally, without tearing the mucosal tissue and also for the lower jaw.

180 mm, 7 1/2"

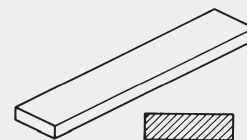
**JG598**

Sterilit i Aesculap®-Schleifstein-Öl

Inhalt: 50 ccm

Sterilit i Aesculap® oil for grinding stones

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Original Arkansas grinding stone, for grinding on tunnelling instruments

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ERGOPLANT  
Spatula, sharp, special  
for soft tissue  
190 mm, 7 1/2"

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DUROTIP-Scissors with inserted carbide cutting edges

- Excellent cutting accuracy
- Long working life
- Gold-plated rings

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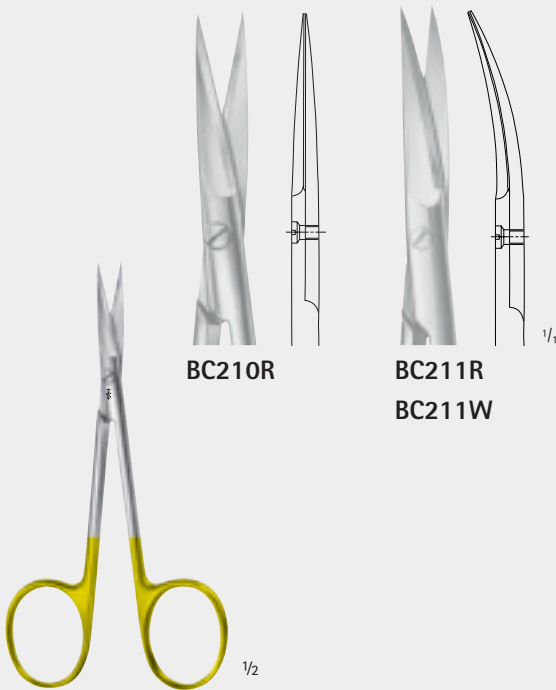
~W-Cut prevents tissue from slipping out

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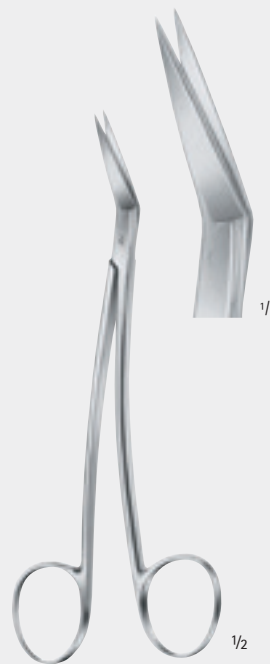
~W-Cut directly after the catalogue number indicates that the resp. item has one serrated cutting edge. (When ordering please indicate ~W-cut together with the catalogue number e.g.: BC211W)

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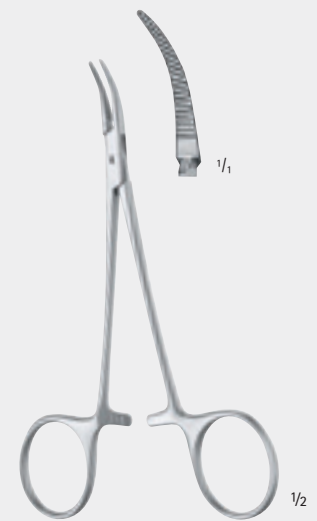
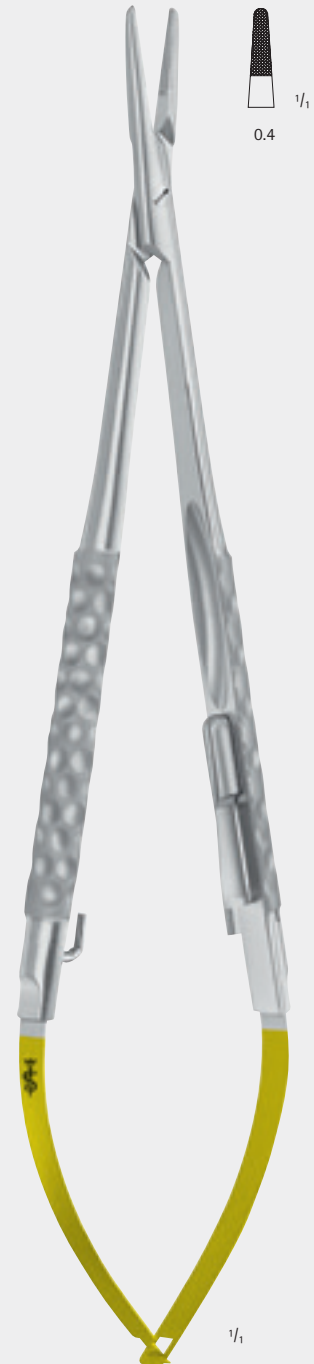
DUROGRIP Needle holder  
for suture material 4/0 – 6/0

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### PARALLEL CLOSING WORKING TIPS

- Enables to grab the soft tissue of the total length of the plateau
- To reduce the contact pressure
- To preserve the delicate soft tissue structures

### THE SIZE OF THE WORKING TIP PLATEAUS

- Enables micro surgery
- Allows a precise grabbing of soft tissue and sutures (until 6/0)
- Mitigates the risk of damaging the soft tissue structures

### SHAPE AND DESIGN

- The waisted shape for a comfortable haptics
- The round handle enables the rotation of the instruments with the tips of the fingers
- Well balanced instruments with a centric balance point

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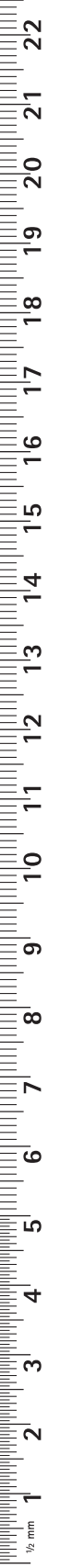
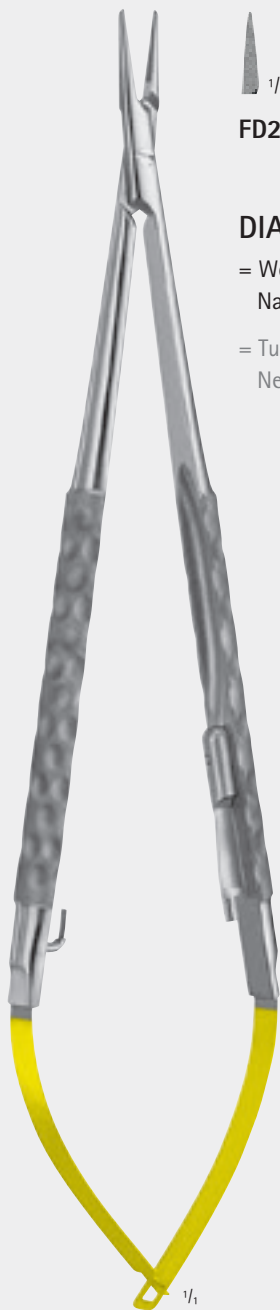
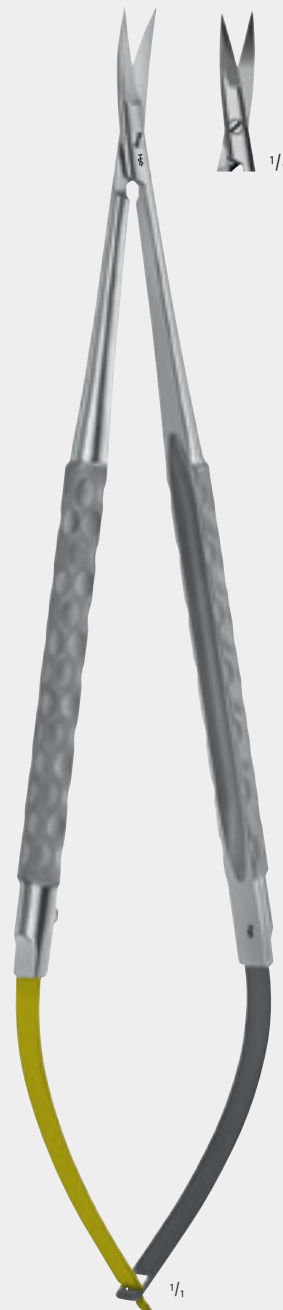
Micro needle holder for suture material 6/0 – 9/0

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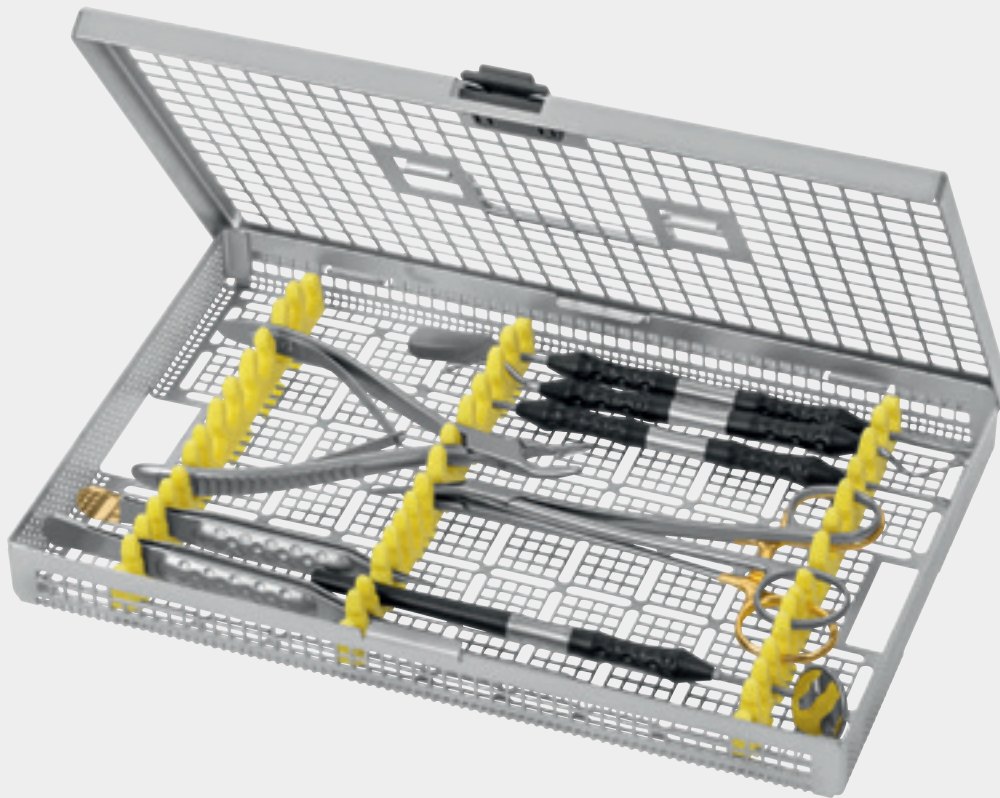
Micro ligature scissors, curved, one blade serrated

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**DX705**

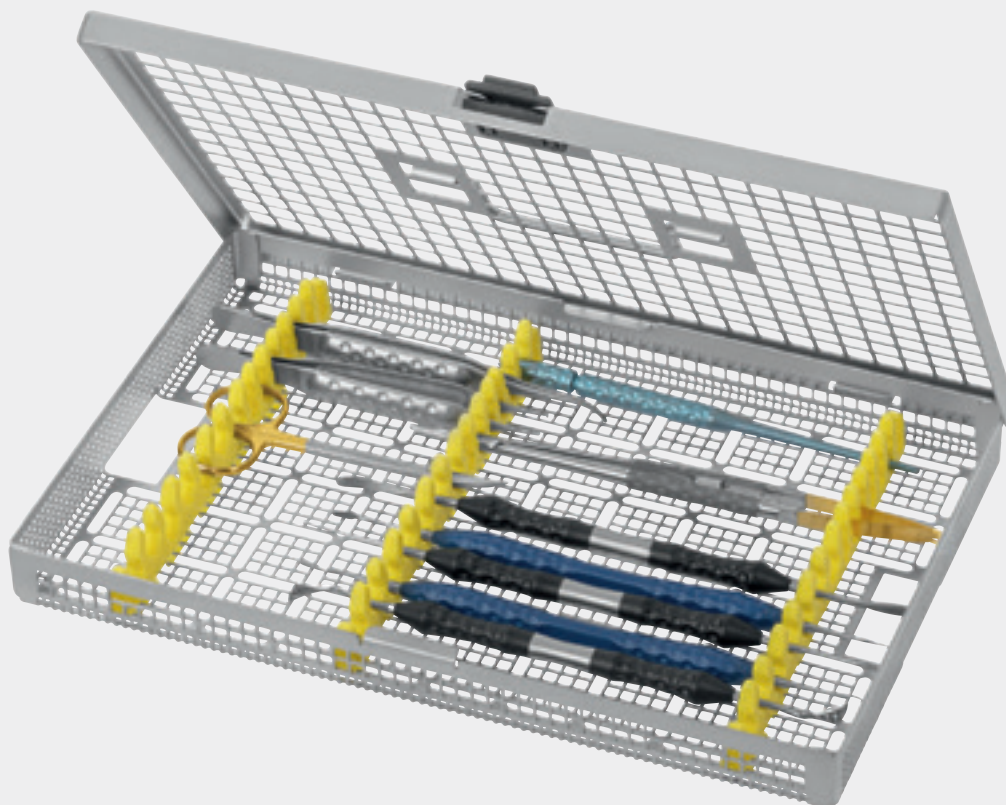
Aesculap® Basic Kit for dental implantology, consisting of:

- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- BB063R 1 Scalpel handle, No. 3
- DX094 1 Ergoperio Mouth mirror holder
- DA036R 1 PAK = Package of 12 Rhodium-Mouth mirror, 22 mm diam.
- DX051R 1 ERGOPLANT Tissue forceps
- DX052R 1 ERGOPLANT Dissecting forceps
- DX201R 1 ERGOPLANT Raspatory, BUSER
- DX200R 1 ERGOPLANT Raspatory, PRICHARD
- DX220R 1 Ergoperio Bone curette, LUCAS
- BM013R 1 DUROGRIP Nadelhalter, BABY CRILE WOOD
- DO219R 1 Gum scissors, LOCKLIN
- FO409R 1 Bone rongeur, LUER-FRIEDMANN
- JG381R 1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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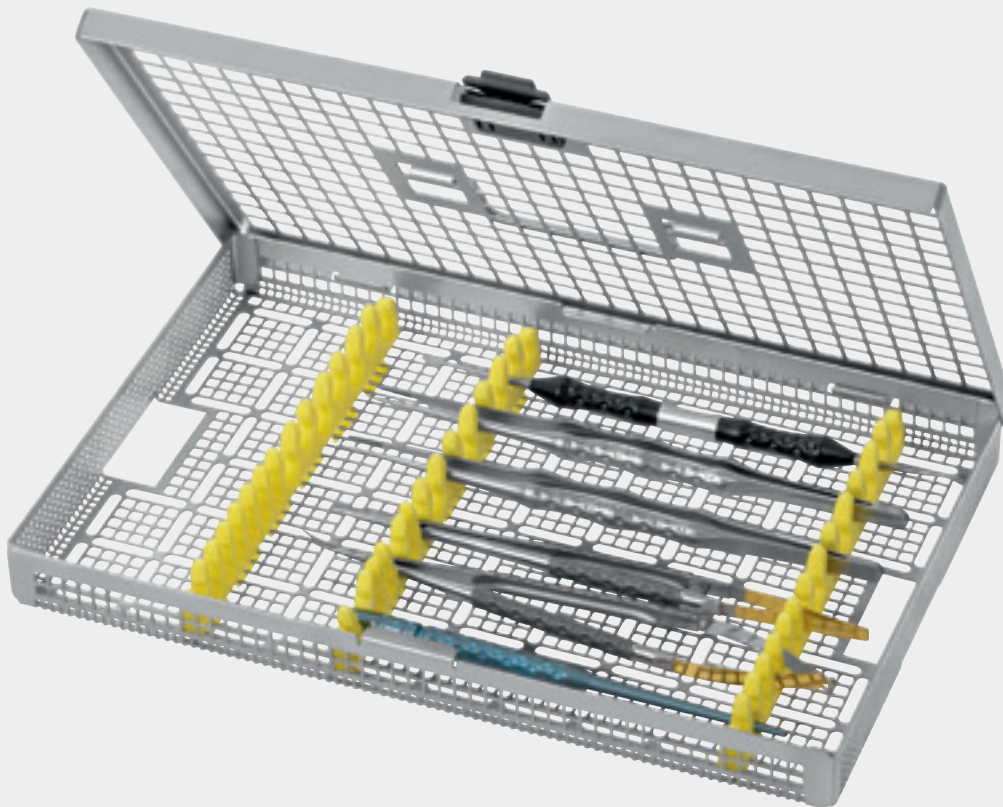
## DX707

Aesculap® Soft Tissue Kit for dental implantology, consisting of:

- BB045T 1 Titanium scalpel handle
- BB367R 1 PAK = Package of 10 micro scalpel blades, sterile, Fig. 367
- BB063R 1 Scalpel handle, No. 3
- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- DX050R 1 ERGOPLANT Tissue forceps
- BM003R 1 DUROGRIP Needle holder
- DX450R 1 ERGOPLANT Gingivectomy knife, KIRKLAND
- DB025R 1 Ergoperio Raspatory FREER
- DX405R 1 ERGOPLANT Atraumatic papilla elevator, PHW
- DX401R 1 ERGOPLANT Spatula, sharp, STRICKER
- DX402R 1 ERGOPLANT Spatula, HEIDEMANN
- BC211W 1 Delicate scissors, curved
- DX055R 1 ERGOPLANT Soft tissue forceps
- JG381R 1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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**DX720**

Micro Soft Tissue Set, consisting of:

- BB045T Titanium-microblade holder, 145 mm
- DX301R Micro-Raspatory, 180 mm
- DX070R Micro atraumata forceps, 180 mm
- FD252R Micro ligature scissors, curved, one blade serrated, 180 mm
- DX060R Micro dissecting forceps, 180 mm
- BM003R DUROGRIP Needle holder, 180 mm
- JG381R Tray, 274 x 172 x 30 mm

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For advanced alveolar crest atrophies less than three millimeters wide it is preferable to pursue a two-stage approach, in which, an autologous bone graft from appropriate autologous donor sites

should be transplanted to the deficient alveolar crest atrophy region prior to the actual implantation. This is exactly what the AESCULAP® Bone Fixation Set was developed for.

# AESCULAP® DENTAL IMPLANTOLOGY

COMPETENCE IN DENTISTRY

## SCHRAUBENMAGAZINE / SCREW MAGAZINE

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with 8 screws, self-taping, cross recess, Ø 1.5 mm

	<u>Länge / length</u>
<b>DX608T</b>	8 mm
<b>DX610T</b>	10 mm
<b>DX612T</b>	12 mm
<b>DX614T</b>	14 mm

## SCHRAUBENMAGAZINE / SCREW MAGAZINE

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with 8 screws, self-taping, cross recess, Ø 1.2 mm

	<u>Länge / length</u>
<b>DX646T</b>	6 mm
<b>DX648T</b>	8 mm
<b>DX650T</b>	10 mm

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### Mechanical cleanable

- The new open magazine storage system enables mechanical cleaning of the implants

### Simple documentation

- The batch numbers are printed on the screw magazines

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**High stability**

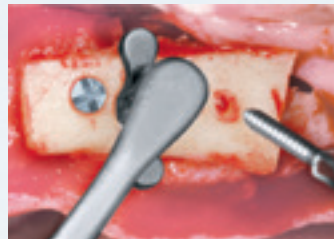
- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy Ti-6Al-4V ensures high stability of the screws

**Small & flat screw head**

- The screw head is easy to countersink in the bone transplant

**Micro thread**

- Allows a good fixation even in very thin residual bone

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Micro screw head = Standard screw head

- Reduces the risk of screw fracturing

**High stability**

- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy allows high stability of the screws

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### Ergonomic

- Reduces the force required for tightening and loosening the screw

### Soft tissue sleeve

- The screw safety sleeve reduces the danger of soft tissue injuries

### Screw fall protection

- The screw safety sleeve serves as a screw fall protection

### Friction

- Enables a strong retention for the screw

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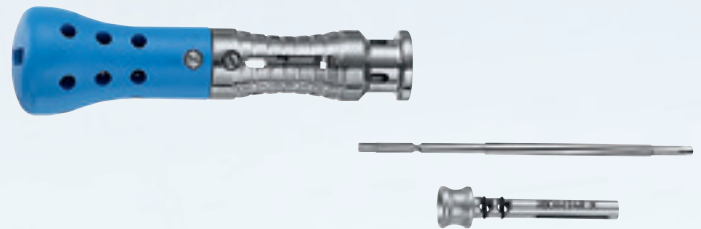
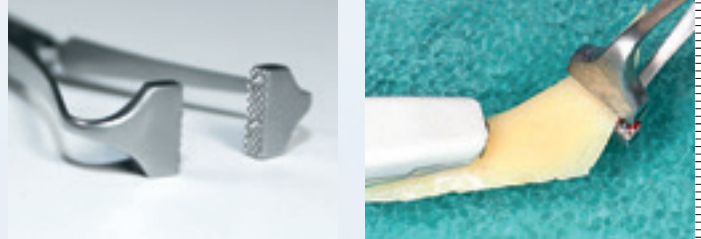
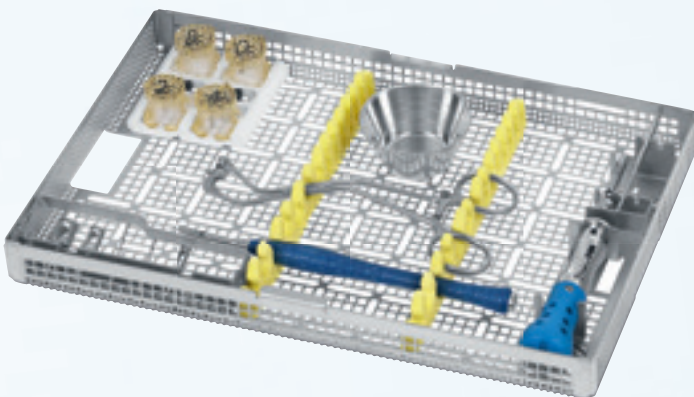
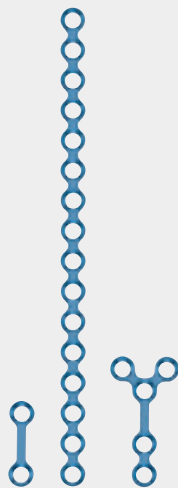
Ø 1.5 mm

Länge / length

<b>DX646T</b>	6 mm	-
<b>DX648T</b>	8 mm	<b>DX608T</b>
<b>DX650T</b>	10 mm	<b>DX610T</b>
-	12 mm	<b>DX612T</b>
-	14 mm	<b>DX614T</b>

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Screw magazine with 8 screws, self-taping, cross recess

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Osteotome, curved, 4 mm wide, for bone block extraction

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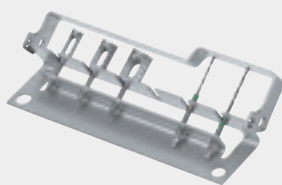
Rack for screw magazines  
(Rack without contents)

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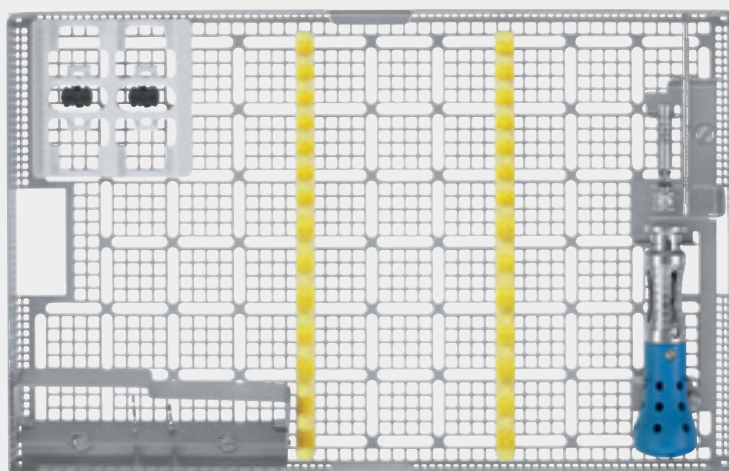
Rack for twist drills  
and tools  
(Rack without contents)

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### DX715

Bone Fixation Set including tray with the  
following contents:

JG403R	Dental tray
DX620	Rack for screw magazines
DX621R	Rack for twist drills
DX630R	Twist drill, Ø 1.25 mm
DX631R	Twist drill, Ø 1.5 mm
DX715200	Rack for screw driver
DX605R	Screw driver
DX606R	Screw driver blade
JG391	Silicone instrument rack (2x)

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Permanent filter for sterile container,  
Order unit: PAK = Pack with 2 pieces

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Autogenous bone in augmentation has been established as the „golden standard“. The region where to gain the bone (Bone Harvesting) and the required consistence for augmentation can vary, depending on the case. Here the ERGOPLANT Micro Bone Mill shows its flexibility whereat a high yield of bone-grist is always ensured.

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#### ERGOPLANT Micro bone-mill

- Small and handy
- Interchangeable cutting disks
- Cutting disks available for finer or coarser bone particles
- Easy to clean due to special cleaning instruments
- Easy mounting and demounting
- High bone yield requires less bone harvesting

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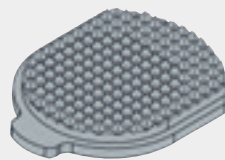
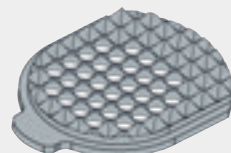
ERGOPLANT  
Micro bone-mill  
(delivered with coarse  
cutting disk DX803R)

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ERGOPLANT  
Cutting disk for DX801R, fine

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ERGOPLANT  
Cutting disk for DX801R, coarse

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Cleaning fork for punching out  
valuable bone material of all cutting  
disk holes at once in a row.

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ERGOPLANT  
Bone-mill-cleaner fine  
for DX802R

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ERGOPLANT  
Bone-mill-cleaner coarse  
for DX803R

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#### DX800

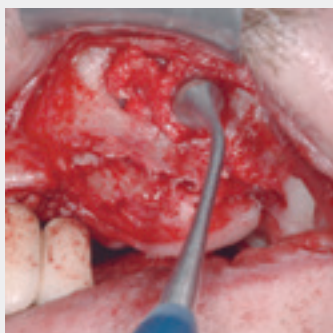
Aesculap® Micro Bone Mill Kit,  
consisting of:

- DX801R 1 ERGOPLANT Micro bone-mill  
(incl. DX803R cutting disk, coarse)
- DX802R 1 ERGOPLANT Cutting disk, fine
- DX810R 1 ERGOPLANT Bone mill-cleaner, fine
- DX811R 1 ERGOPLANT Bone mill-cleaner, coarse
- JF284R 1 Tray (274 x 172 x 60 mm)  
with storage aid for micro bone-mill,  
suitable for dental container JN095
- TE888 1 Packing stencil (only available for DX800)

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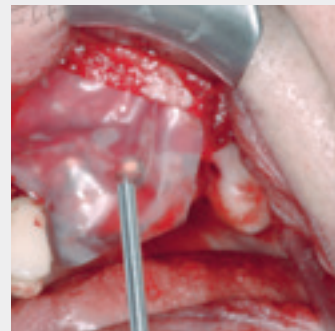
Enables secure carrying and placing of enough bone material. The other working tip serves for pushing and plugging bone material into the augmentation area.

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**ERGOPLANT**  
Applicator for  
bone material

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The small surface of the pointed tip avoids the sticking of the humid membrane. The small spatula on the other end, helps to take out bone material of, for instance, the bone files DX252R – DX253R, as shown on page 42.

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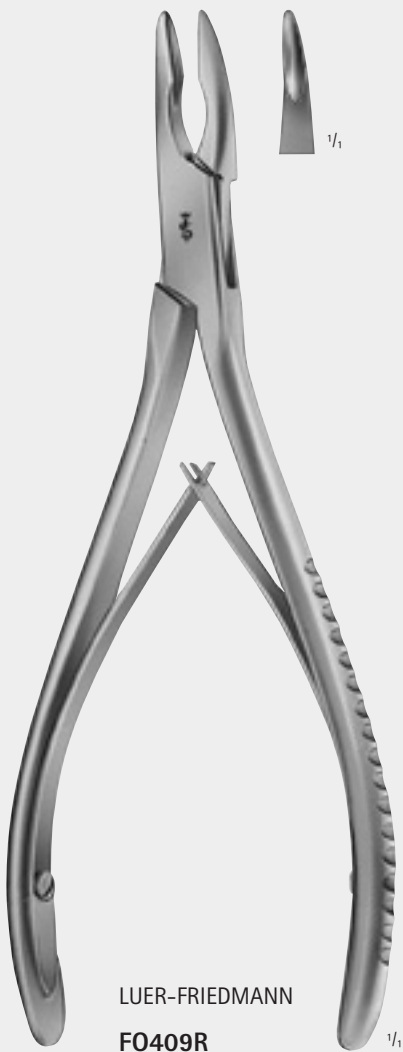
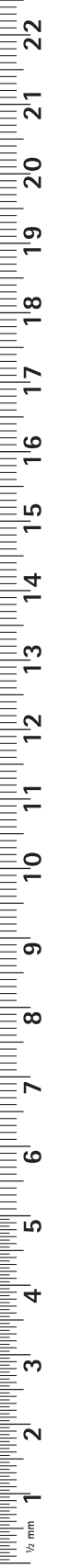
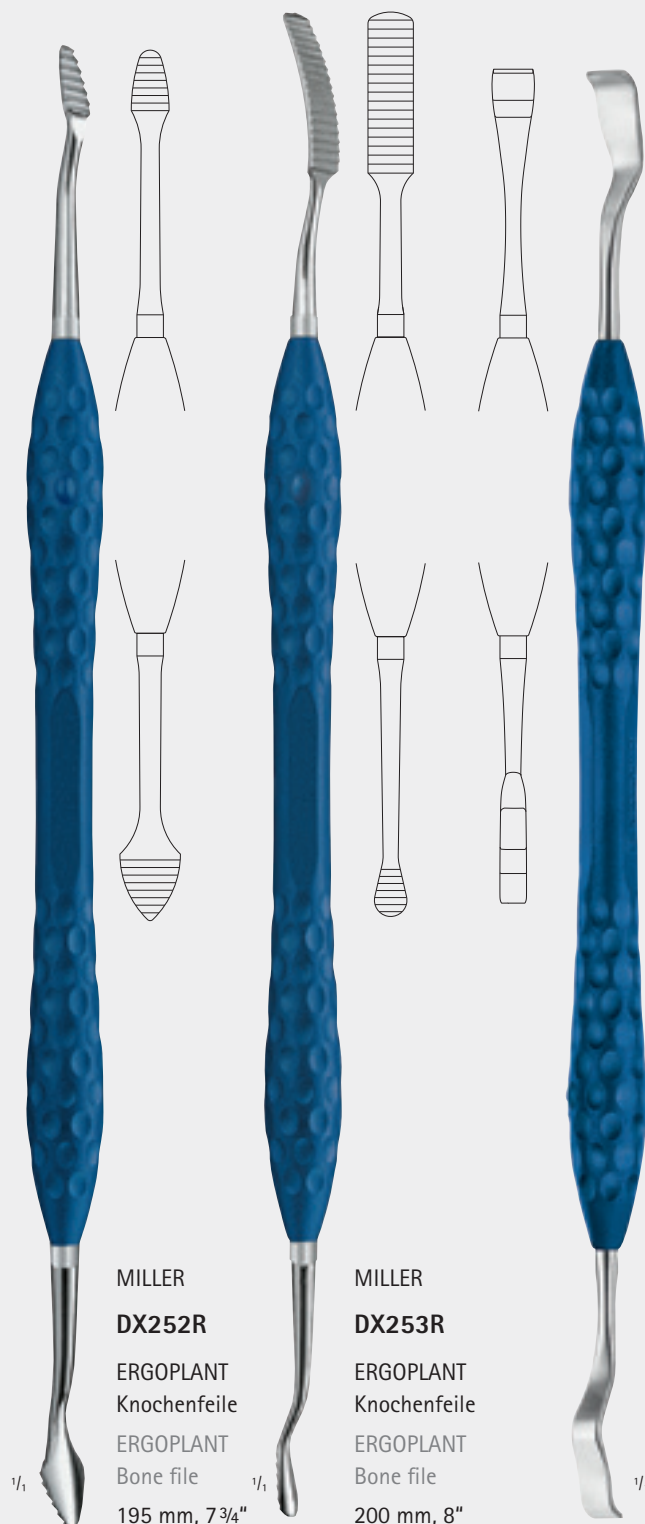
If smaller quantity of bone material is required, with the bone files DX252R – DX253R it can be harvested directly from the jawbone, without additional trauma for the patient. The quantity is often sufficient, for augmentation around the implant-neck.

**MILLER**  
**DX252R**  
ERGOPLANT  
Knochenfeile  
ERGOPLANT  
Bone file  
195 mm, 7 3/4"

**MILLER**  
**DX253R**  
ERGOPLANT  
Knochenfeile  
ERGOPLANT  
Bone file  
200 mm, 8"

**BUSER**  
**DX254R**  
ERGOPLANT  
Knochenschaber, 3 / 5 mm  
ERGOPLANT  
Bone raspatory, 3 / 5 mm  
190 mm, 7 1/2"

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With this raspatory fine bone particles can be harvested by simply scratching over the jawbone.

ERGOPLANT – KNOCHENGEWINNUNG / BONE SPLIT  
 ERGOPLANT – BONE HARVESTING / BONE SPLIT

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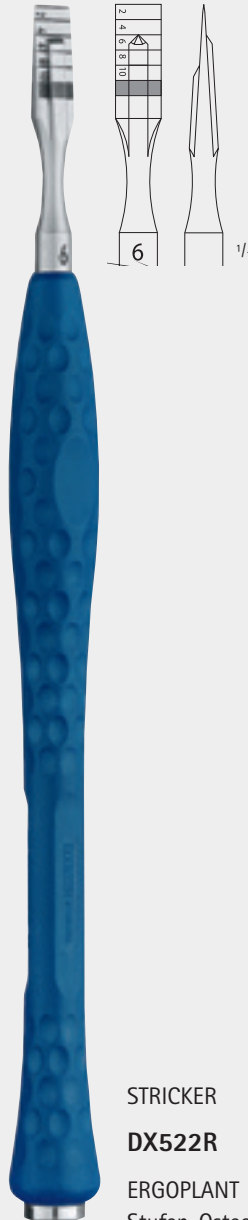
big laser marking after 10 mm as security zone

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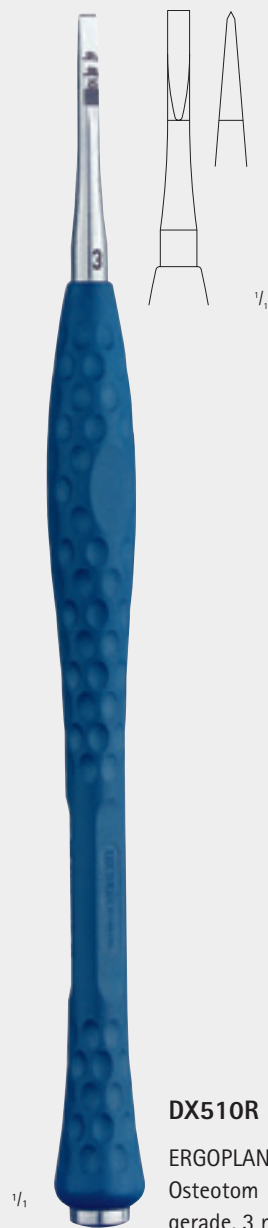
The Step-Osteotome was specially developed for secure bone-splitting with simultaneous stepwise widening. Due to the specially designed working tips, the maxillary crest will be first expanded towards palatinal and only afterwards towards vestibular. In the same time spongy bone in the center will be condensed. After removing the Step-Osteotome, a pre-selection point for the implant bur can be clearly seen.

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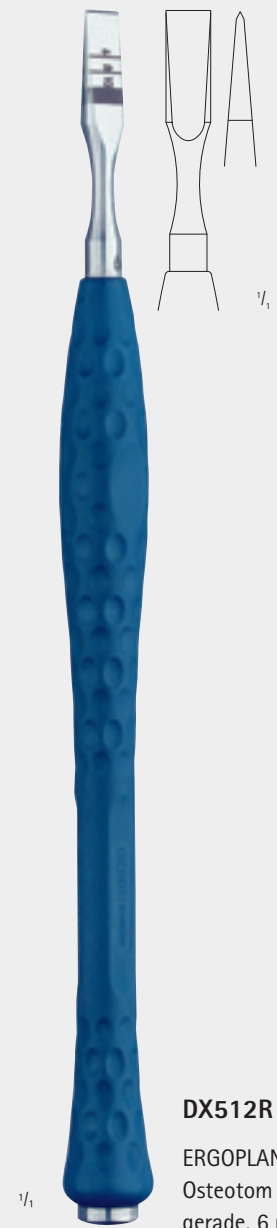
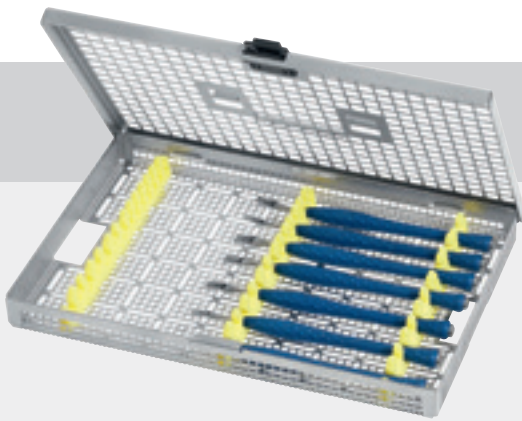
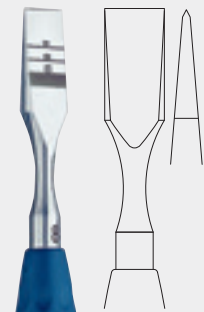
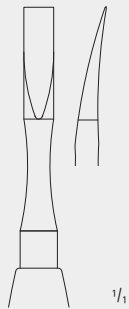
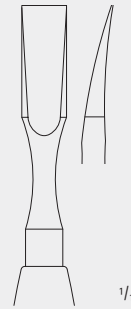
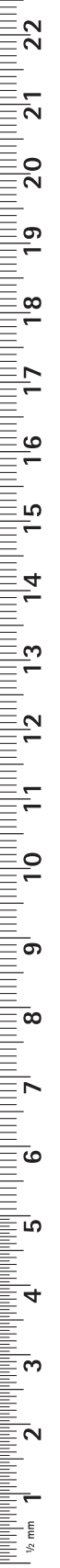
## DX708

Aesculap® Bone Split Kit for dental implantology,  
consisting of:

DX510R 1 ERGOPLANT Osteotome, straight, 3 mm  
DX511R 1 ERGOPLANT Osteotome, straight, 4 mm  
DX512R 1 ERGOPLANT Osteotome, straight, 6 mm  
DX513R 1 ERGOPLANT Osteotome, straight, 8 mm  
DX517R 1 ERGOPLANT Osteotome, curved, 4 mm  
DX518R 1 ERGOPLANT Osteotome, curved, 6 mm  
DX522R 1 ERGOPLANT Step-Osteotome, 6 mm  
JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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The maxillary crest is splitted transversal by the Step-Osteotome.

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The bone split is kept apart by the retractor.

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The implant can be positioned by using the existing bone.

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After splitting the maxillary crest the bone split is closing within a short period of time. This leads to an unwished and unnecessary bone loss when drilling afterwards. This causes the risk of an implant that is not exactly positioned and additional augmentative treatments could be necessary. The bone split is kept apart by the retractor. Thus, the existing bone can be, used better to position the implant.

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
**DX709**

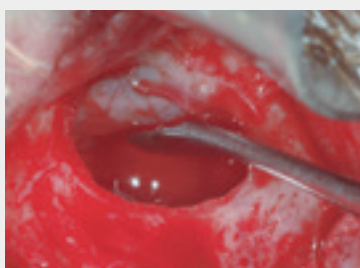
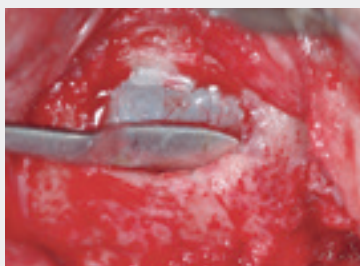
Aesculap® Bone Condenser Kit for dental implantology, consisting of:

- DX530R 1 ERGOPLANT Bone condenser, straight, 1.5 – 2.2 mm
- DX531R 1 ERGOPLANT Bone condenser, straight, 2.2 – 2.8 mm
- DX532R 1 ERGOPLANT Bone condenser, straight, 2.8 – 3.5 mm
- DX533R 1 ERGOPLANT Bone condenser, straight, 3.5 – 4.2 mm
- DX534R 1 ERGOPLANT Bone condenser, straight, 4.2 – 4.8 mm
- DX535R 1 ERGOPLANT Bone condenser, bayonet, 1.5 – 2.2 mm
- DX536R 1 ERGOPLANT Bone condenser, bayonet, 2.2 – 2.8 mm
- DX537R 1 ERGOPLANT Bone condenser, bayonet, 2.8 – 3.5 mm
- DX538R 1 ERGOPLANT Bone condenser, bayonet, 3.5 – 4.2 mm
- DX539R 1 ERGOPLANT Bone condenser, bayonet, 4.2 – 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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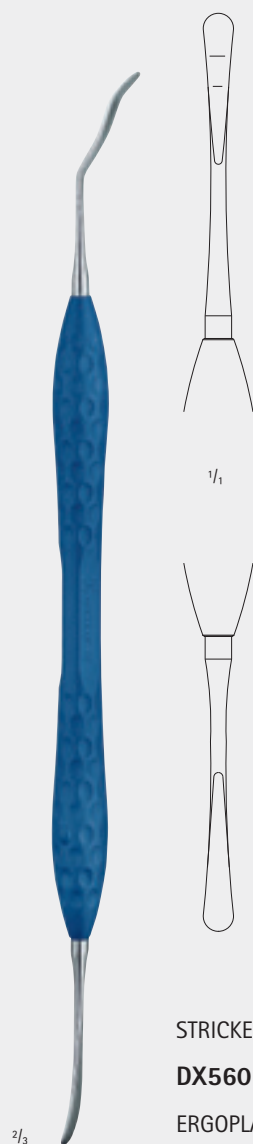
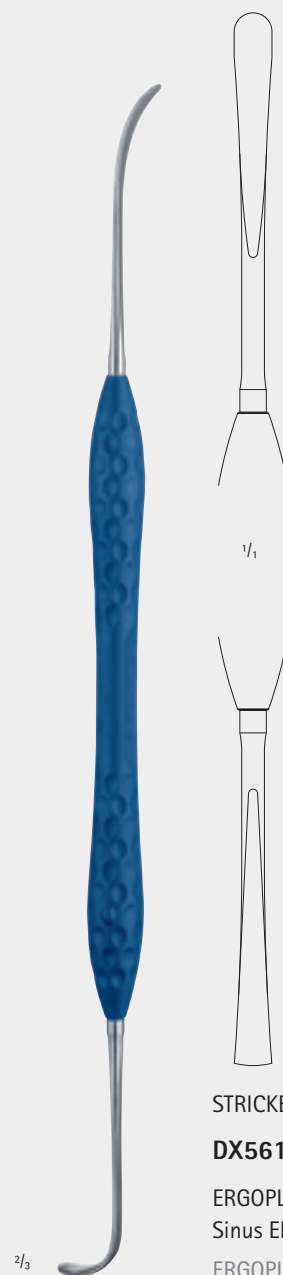
Separating the maxillary sinus membrane carefully from the bony base. Finally the sinus window will be pushed with the other working tip gently into the hollow maxillary cave.

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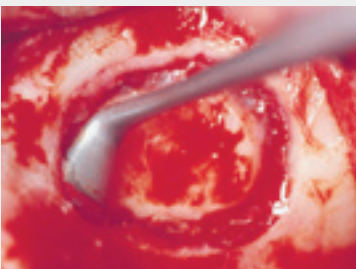
The Schneiderian membrane of the sinus can be detached with elevator D0571R by 1 – 2 mm along the total cortical bone.

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Elevator D0573R is particularly useful in the lower part for detaching the membrane further. The same goes for the caudal area, especially when the membrane is 4 – 5 mm above the base of the maxillary sinus and the maxillary sinus is built up with granular material.

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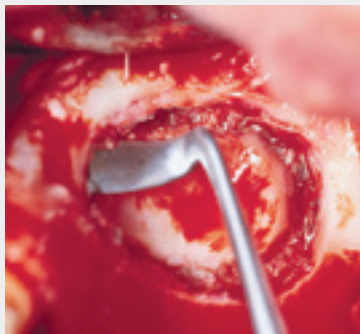
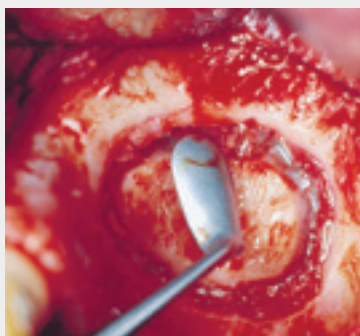
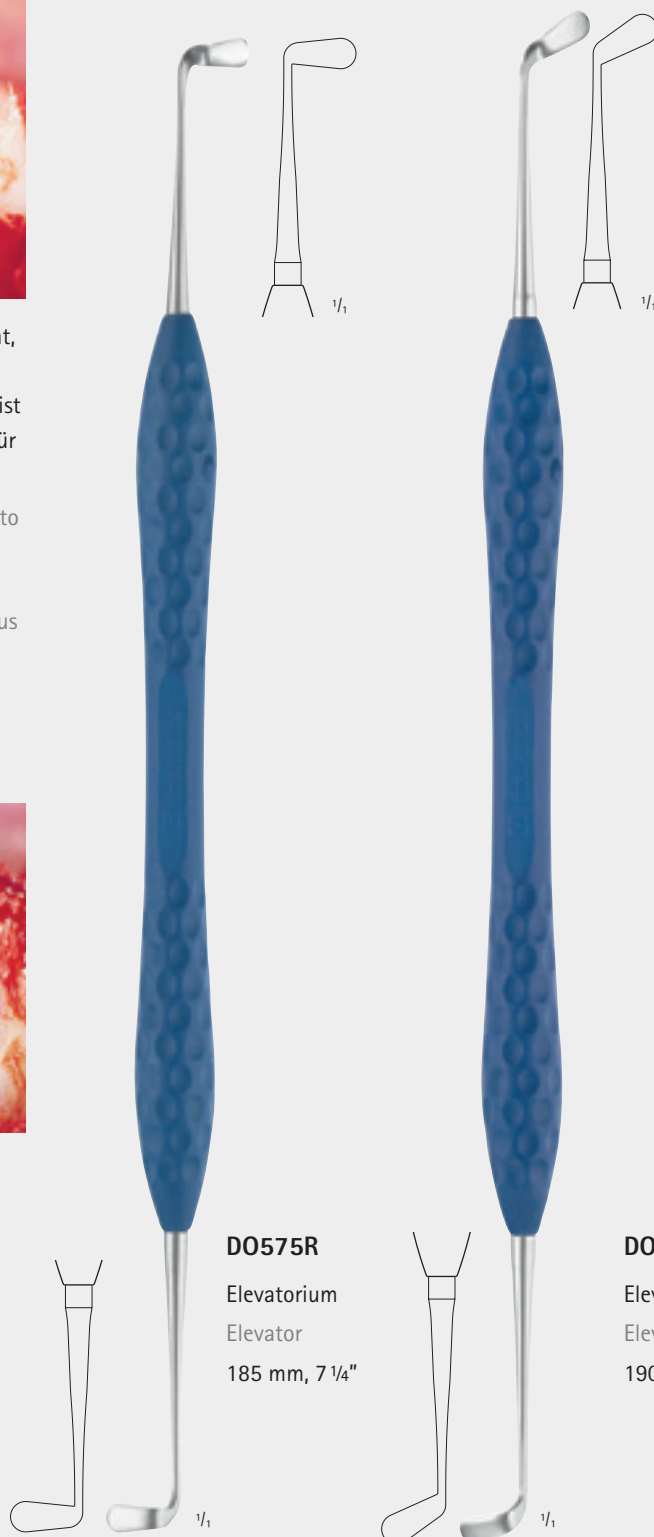
Elevator D0575R makes it possible to lift the membrane further in mesial region. One working tip is specially designed for the right maxillary sinus and the other for the left maxillary sinus.

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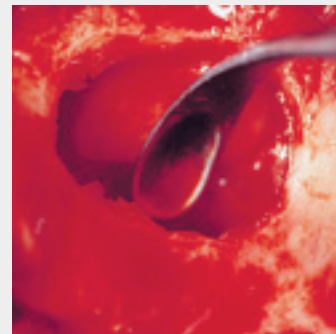
With elevator D0574R, the sinus membrane is further elevated in the upper and posterior part of the maxillary sinus wall.

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With the aid of elevator D0572R, the sinus membrane is further detached over a broad area in the inner region, especially along the roof of the palate. Condenser DX555R permits positioning of the filling material.

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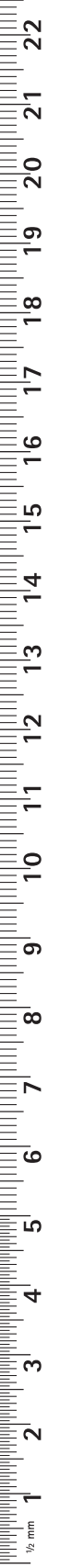
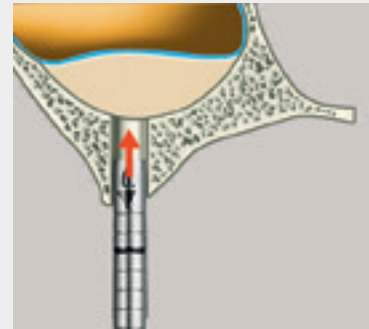
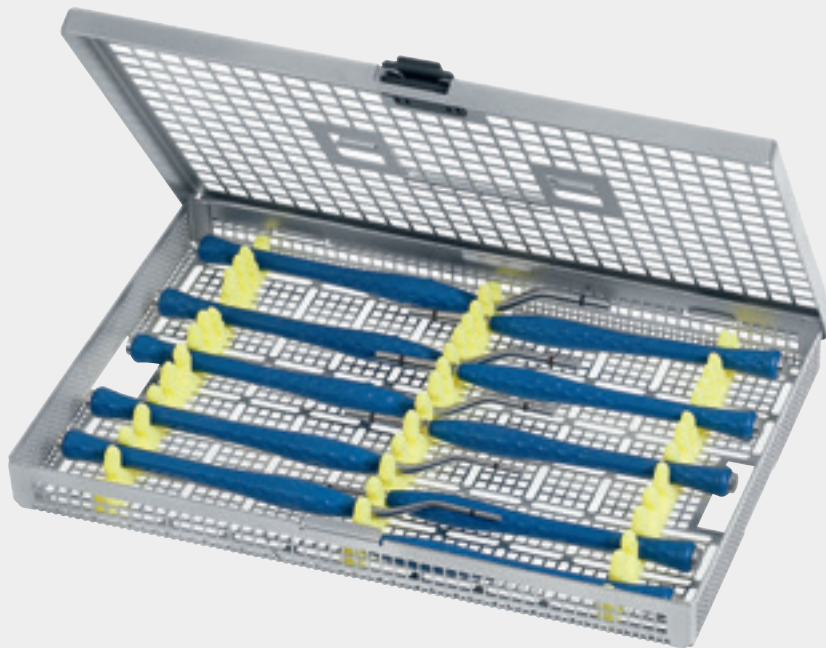
This illustration shows the technique  
of the percrestale sinus elevation.

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## DX710

Aesculap® Sinutome Kit for dental implantology, consisting of:

- DX570R 1 ERGOPLANT Sinutome, straight, Ø 2.2 mm
- DX571R 1 ERGOPLANT Sinutome, straight, Ø 2.8 mm
- DX572R 1 ERGOPLANT Sinutome, straight, Ø 3.5 mm
- DX573R 1 ERGOPLANT Sinutome, straight, Ø 4.2 mm
- DX574R 1 ERGOPLANT Sinutome, straight, Ø 4.8 mm
- DX575R 1 ERGOPLANT Sinutome, bayonet, Ø 2.2 mm
- DX576R 1 ERGOPLANT Sinutome, bayonet, Ø 2.8 mm
- DX577R 1 ERGOPLANT Sinutome, bayonet, Ø 3.5 mm
- DX578R 1 ERGOPLANT Sinutome, bayonet, Ø 4.2 mm
- DX579R 1 ERGOPLANT Sinutome, bayonet, Ø 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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- Very delicate working tips facilitate access to dental implant collar
- No risk of infection from curette abrasion
- Low implant damage risk
- Manually sharpenable

To ensure the long-term success of an implant, consistent aftercare is very important.

To eliminate plaque on the implant-abutment interface, titanium alloy currettes can be preferable.

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Force-controlled periodontal probe

- Defined probing force of 0.2 N (20 g)
  - Low implant damage risk
- Titanium aluminium nitride coating
  - Exact marking and smooth transition
  - No abrasion

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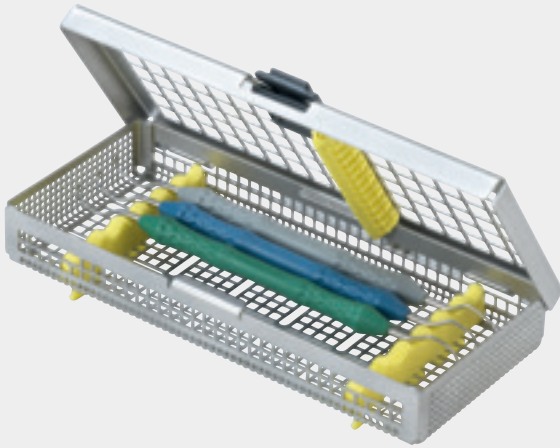
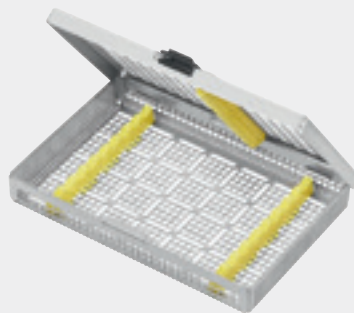
Recall Set consisting of DB180T, DB181T, DB182T and JG387R

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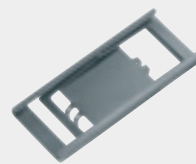
Holding device for identification labels

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Plastic identification labels, white, with 10-digit barcode for praxiDoc or DIOS software applications, labeling as specified

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These instruments were developed to hold back lips, cheeks, tongue and lingual mucous membrane. Due to the ergonomic handle design, a more relaxed working with minor physical strain is possible, even in long operations.

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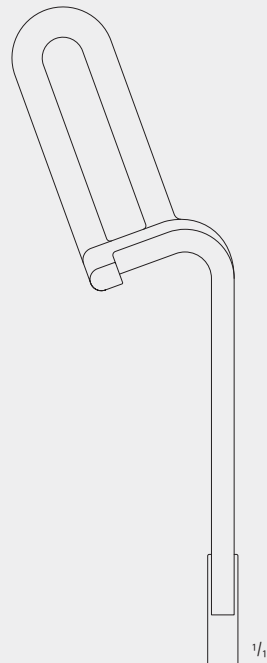
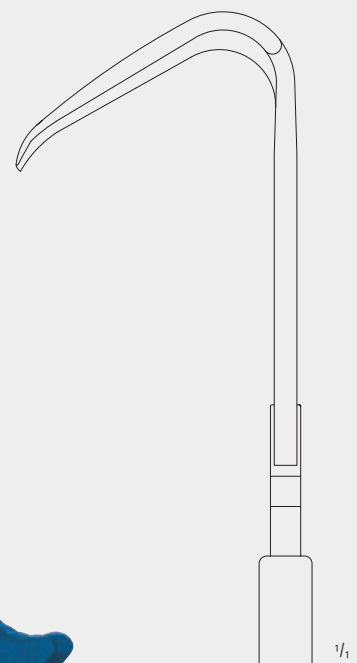
The mouth wedges are particularly advantageous for longer implantations and also provide support for the patient.

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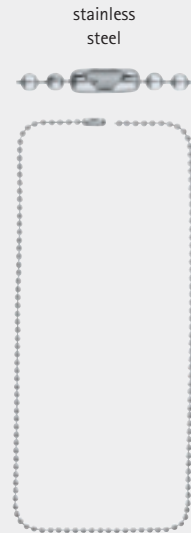
Sales unit: PAK = Package of 2 pieces  
 suitable for steam sterilization 134°C  
 (Autoclav)

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ERGOPLANT  
 Silicone mouth wedge  
 Size: S

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ERGOPLANT  
 Silicone mouth wedge with metal core  
 Size: M

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ERGOPLANT  
 Silicone mouth wedge with metal core  
 Size: L

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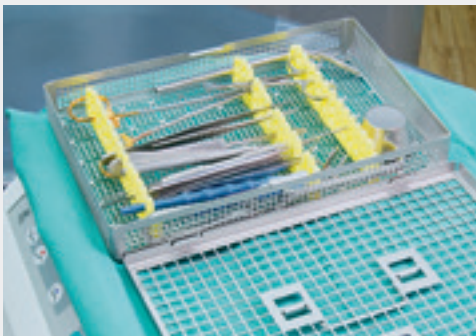
## Systematic Instrument Reprocessing Procedure

The primary intention of hygienic measures in dental practice is to safeguard the health of patients and personnel. To ensure safety it is particularly important to observe the principles of hygiene when cleaning and sterilizing instruments.

To maintain a safe, efficient and economical workplace, clear procedures for hygiene must be specified and followed. The Aesculap® Sterile Container System is a proven and excellent aid in the standardized management of instruments throughout the dental practice workflow, from sterile processing through transport, storage and use.

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## Four smart reasons to choose the Aesculap® Sterile Container System

- **Prevention of infections**  
Of patients and staff through preservation of instrument sterility and closed transport of used instruments
- **Economy**  
Low total cost of ownership due to reusable packaging, minimal processing costs and long service life
- **Comfortable handling**  
Simple, quick and easy
- **Transparency and reduction of errors**  
Standardized, effective documentation of your sterile supply workflow

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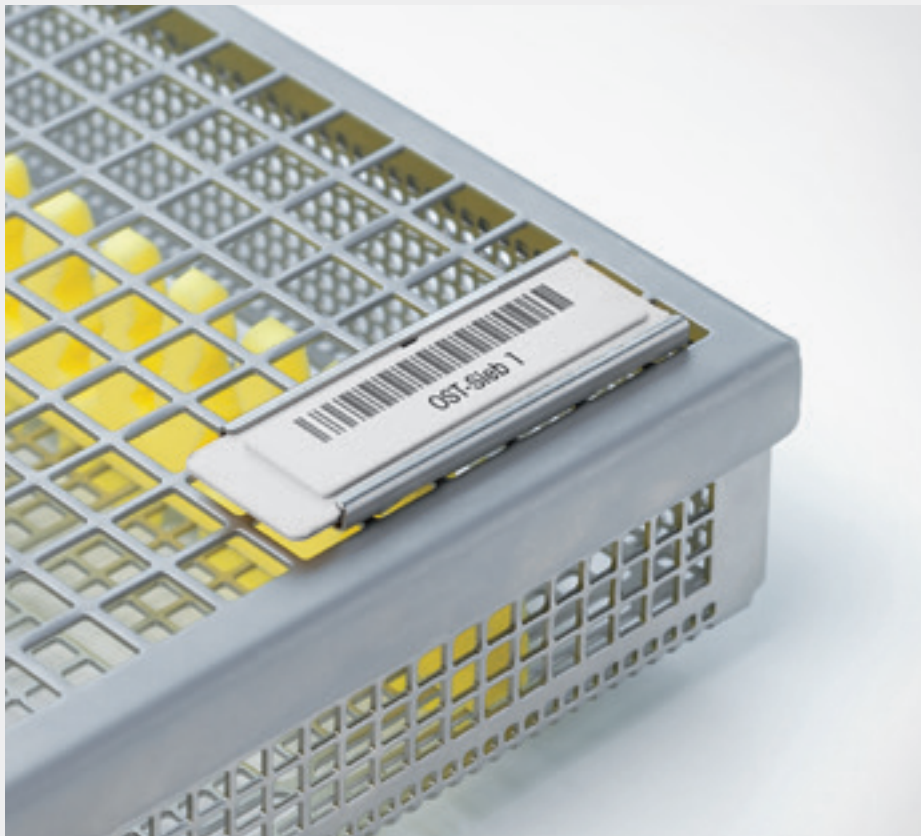
## Effective cleaning

The wide perforation ensures optimal water jet permeation and avoids rinsing blind spots.

*„The microbiological and organic contamination is satisfactorily reduced“*

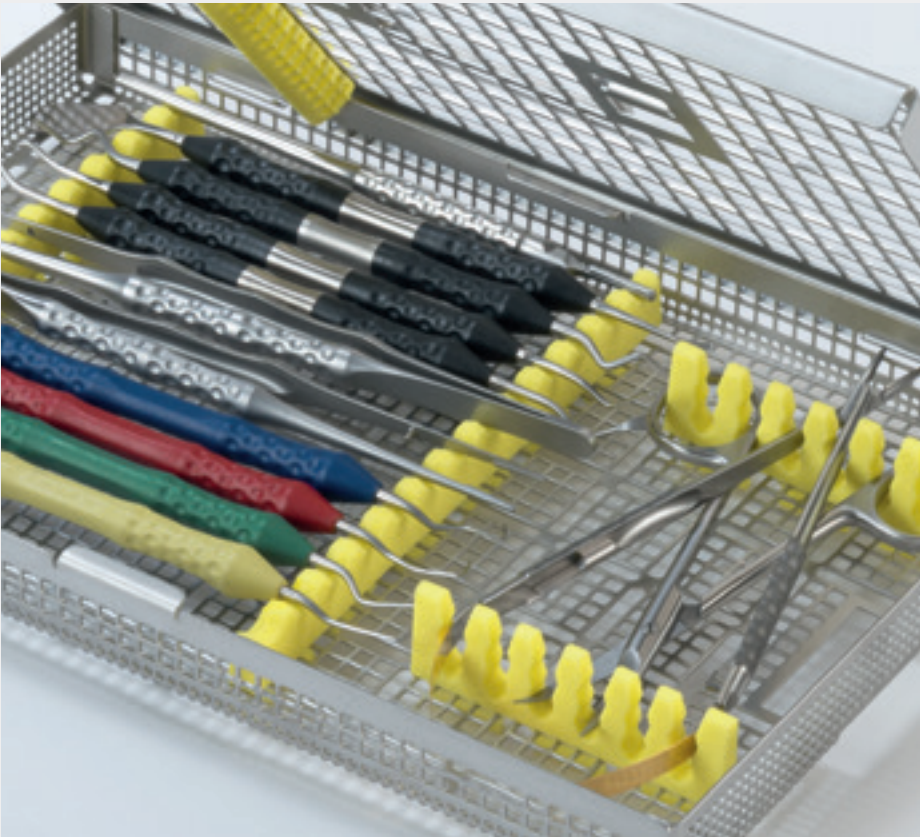
This is the result of extensive testing carried out by the independent laboratory Medical Device Testing.

PAR surgical instrument sets in Aesculap® trays contaminated with test germs and organic matter were used for testing.

Consequently, the mechanical cleaning and thermal disinfection before sterilization were rated as being adequate in accordance with DIN EN ISO 17664.

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## Perfect organization

Your instruments are stored in the smallest possible space, safely fixed in their positions.

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## Practical and economical

Mechanical processing with the Aesculap® tray system optimizes your workflow. The stainless steel material makes Aesculap® trays a robust and durable asset. The simple and robust locking system can be easily opened and closed with one hand.

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## Flexible and adaptable

Position the silicone racks in the optimal arrangement for your instruments – in seconds, without any additional tools.

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Containers with base perforation  
(for Steam sterilization through fractionated vacuum process)

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To take following trays:  
1 x JG381R or 1 x JG383R or 1 x JG389R  
JK286 container body and JN091 lid can  
also be ordered singly  
312 x 190 x 46 mm

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To take following trays:  
2 x JG380R or 1 x JG384R or 1 x JG386R  
JK288 container body and JN091 lid can  
also be ordered singly  
312 x 190 x 65 mm

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To take following trays:  
2 x JG381R or 1 x JG381R + 1 x JG384R or  
1 x JG285R  
JK289 container body and JN091 lid can  
also be ordered singly  
312 x 190 x 92 mm

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To take all trays or 1 x JF282R basket with lid  
JF289R  
JK290 container body and JN091 lid can also be  
ordered singly  
312 x 190 x 130 mm

Verbrauchsmaterialien bitte separat bestellen. Please order accessories separately.

CONTAINER MIT DECKELPERFORATION, DENTALSIEBTRAYS  
CONTAINERS WITH BASE PERFORATION, DENTAL TRAYS

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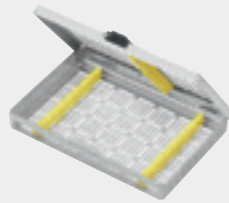
Tray, contains two instrument racks (2 x JG394) for 6 instruments

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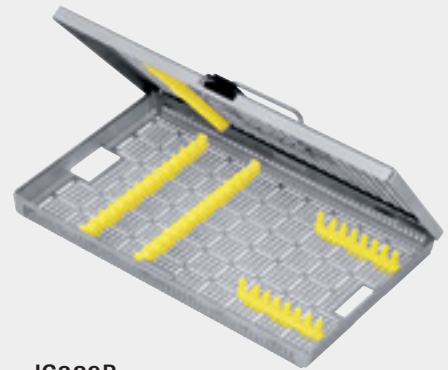
Tray, contains two instrument racks (JG400 + JG401) for 9 instruments

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Tray, contains five instrument racks (2 x JG394, 2 x JG400, 1 x JG401), with lid handle, fitting into Dental Container JN092

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Tray, contains three instrument racks (3 x JG391), with lid handle, fitting into Dental Container JN092

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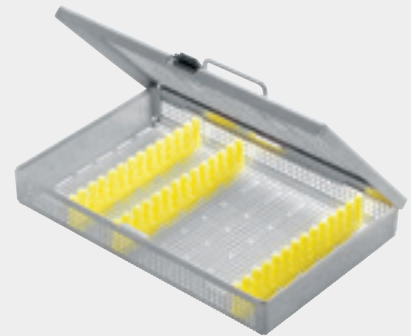
Tray, with perforation for retractor, contains three instrument racks (3 x JG391), with lid handle, fitting into Dental Container JN092

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Tray, contains three instrument racks (3 x JG393), with lid handle, fitting into Dental Container JN094

273 x 176 x 41 mm (hoch)

# AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 7461 95-0 | Fax +49 7461 95-2600 | [www.aesculap-dental.com](http://www.aesculap-dental.com)

The main product trademark "Aesculap" and the product trademarks "DIADUST", "DUROGRIP", "DUROTIP" and "Sterilit" are registered trademarks of Aesculap AG. Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.