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GB Distributed by
B. Braun Medical Ltd.
Sheffield, S35 2PW
United Kingdom

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AU Distributed by
B.Braun Australia Pty. Ltd
Bella Vista NSW 2153 – Australia

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CZ Distributor
B. Braun Medical s.r.o.
148 00 Praha 4 – Czech Republic

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US Distributed by:
B. Braun Medical Inc.
Bethlehem, PA 18018-3524 – USA

NO812 / 2021-08

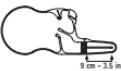
B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud | FRANCE

B|BRAUN

ACTREEN®
Mini Set

B|BRAUN

EN Instructions for use

Actreen® Mini Intermittent Urinary Catheters are indicated for intermittent urinary catheterisation by female patients with chronic urine retention or voiding dysfunction.
Actreen® Mini Set is a sterile, pre-lubricated, ready-to-use catheter connected to a collecting bag equipped with an anti-reflux valve. The lubricant is water and glycerin based.

For safety and hygienic reasons, we advise you to use the product in shortest possible time after opening

Unfold the set by removing the white sticker on the back. **A**

Preparation:

- 1 - Carefully wash hands before and after catheterisation, and clean the area surrounding the urethral opening.
- 2 - Before opening the catheter, check that the pouch is intact and inspect it (visually and by touch), also check the catheter to ensure the lubricant is spread evenly.
- 3 - Open the catheter by tearing and removing the pre-cut protective sheath. **B**
- 4 - Hold the catheter by the connector without touching the catheter.

Catheterisation:

- 5 - With one hand, separate the labia and with the other, gently insert the catheter through the urethral opening into the urethra **C** and up into the bladder, until urine starts to flow into the bag.
- 6 - Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder is completely emptied.
- 7 - When the bladder is completely empty, gently withdraw the catheter.

Emptying:

- 8 - To empty the bag, tear off along the dotted line located on the top. **D**
To discreetly dispose of Actreen® Mini Set, refold the set and reposition the sticker to close it. **E**

Then discard the catheter according to the current local regulations.

! Self-catheterisation should only be carried out under medical advice and only in accordance with the instructions given.

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